



HOW TO PROGRAMME/PAIR WIRELESS PAD TRANSMITTER (WPT-01)

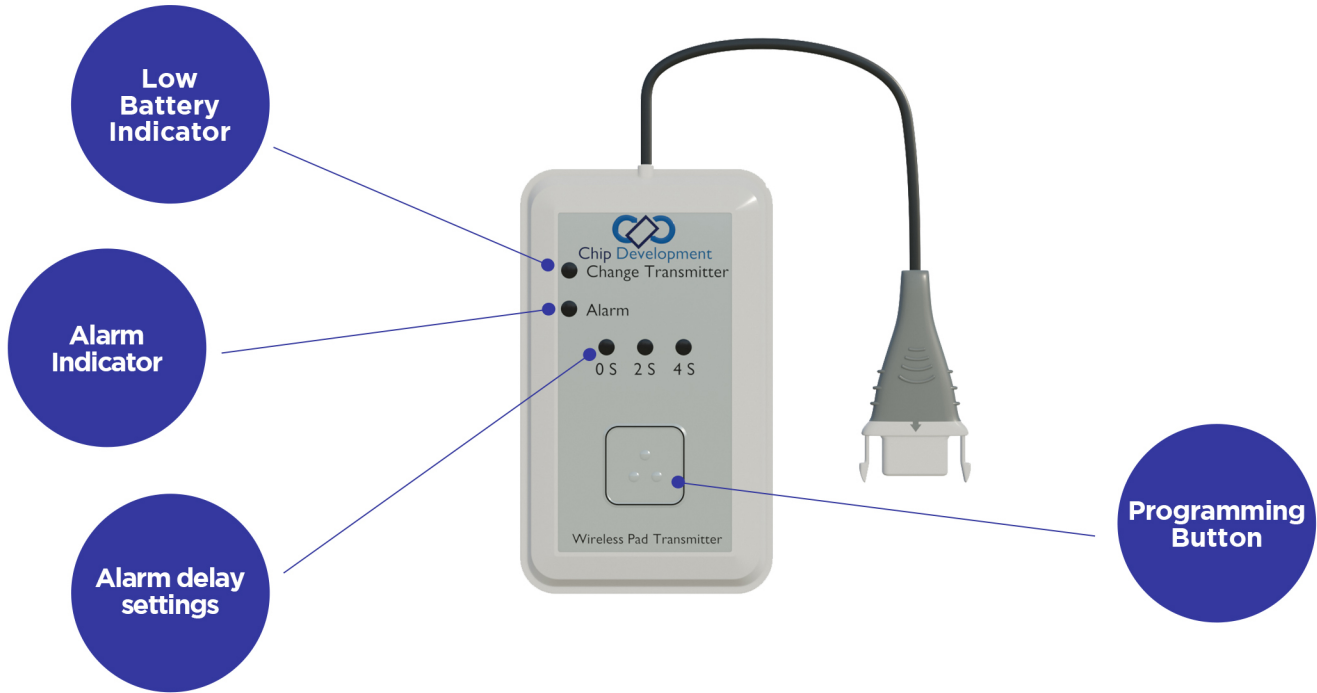

Wireless Pad Transmitter can be connected to all Chip Development Chair, Bed and Under Mattress Sensor Pads. This transmitter can be wirelessly paired with NCR, Pager or Wireless Falls Monitor.

<p>Programme with NCR (Nurse Call Receiver)</p> 	<ul style="list-style-type: none"> • Step 1: Insert 2 x AA Alkaline batteries into NCR. • Step 2: Make sure Transmitter (WPT-01) wire is not connected/plugged with sensor pads. • Step 3: Also make sure the Transmitter is out of sleep mode. To do that, press and hold the button located in the middle of the transmitter for up to 10 seconds until both LEDs flash. • Step 4: Bring the transmitter close to NCR. Flick programming switch at NCR from W to P. Channel 1 starts flashing. • Step 5: Use the “Reset Button” at the front bottom of NCR to change and select the channel you want to use. • Step 6: Press and hold the button located in the middle of the transmitter for up to 10 seconds until you hear a beep from NCR and the flashing LED has moved to the next channel. When you do not hear the beep, release the button at Transmitter and press and hold it again for over 10 seconds. • Step 7: Switch the programming switch back to W. • Step 8: Plug transmitter into Chair/Bed or under mattress sensor pads for testing.
<p>Alarm delay settings</p>	<ul style="list-style-type: none"> • Step 1: Make sure Transmitter is plugged with Sensor Pad and Pressure/weight is applied. • Step 2: Press and hold Programming button at Transmitter WPT-01 for few seconds and then select delay option you want. 0S for instant, 2 seconds and 4 Seconds.

HOW TO PROGRAMME/PAIR WIRELESS PAD TRANSMITTER (WPT-01)

Programme with PGR-01 (Wireless Pager)



- **Step 1:** Insert 2 x AA Alkaline batteries into Wireless Pager
- **Step 2:** Make sure Transmitter (WPT-01) wire is not connected/plugged to sensor pads.
- **Step 3:** Also make sure the Transmitter is out of sleep mode. To do that, press and hold the button located in the middle of the transmitter for up to 10 seconds until both LEDs flash.
- **Step 4:** Press and Hold both Up & Down buttons simultaneously until programming icon “P” starts flashing.
- **Step 5:** Use Up, Down and Reset button to assign 4 digit number for transmitter. Such as Room Number.
- **Step 6:** While “P” is flashing, Press and hold the button located in the middle of the transmitter for up to 10 seconds until you hear a beep from Pager. When you do not hear the beep, release the button at Transmitter and press and hold it again for over 10 seconds.
- **Step 7:** Get out from Programming mode by pressing and holding Up & Down button until icon “P” stops flashing
- **Step 8:** Plug transmitter into Chair/Bed or under mattress sensor pads for testing.

Programme with WFM-01 (Wireless Falls Monitor)



- **Step 1:** Insert 1 x 9V battery into Wireless Falls Monitor WFM-01
- **Step 2:** Make sure Transmitter (WPT-01) wire is not connected/plugged to sensor pads.
- **Step 3:** Also make sure the Transmitter is out of sleep mode. To do that, press and hold the button located in the middle of the transmitter for up to 10 seconds until both LEDs flash.
- **Step 4:** Press and Hold “Hold/Reset Button” located at top left side of Monitor. Channel “A” will start flashing.
- **Step 5:** Use Hold button again to select Channel A or B for the pairing.
- **Step 6:** While A or B is flashing , Press and hold the button located in the middle of the transmitter for up to 10 seconds until you hear a beep from Monitor. When you do not hear the beep, release the button at Transmitter and press and hold it again for over 10 seconds.
- **Step 7:** Get out from Programming mode by pressing Hold button until Channel A and B stops flashing
- **Step 8:** Plug transmitter into Chair/Bed or under mattress sensor pads for testing.